



Thompson House
Unit 10 Styles Close, Sittingbourne, Kent, ME10 3BF
Tel: 01795 477280 Fax: 01795 229692
E-mail address: sales@rms-kent.co.uk
www.ineedawheelchair.co.uk





Dec 2021


GACRK-IFU-2021-rev0



Grillo Ankle Cuff Rail kit

 Tools needed: 10mm spanner, 4mm hex key

 This kit should only be fitted by a suitably qualified Healthcare professional.

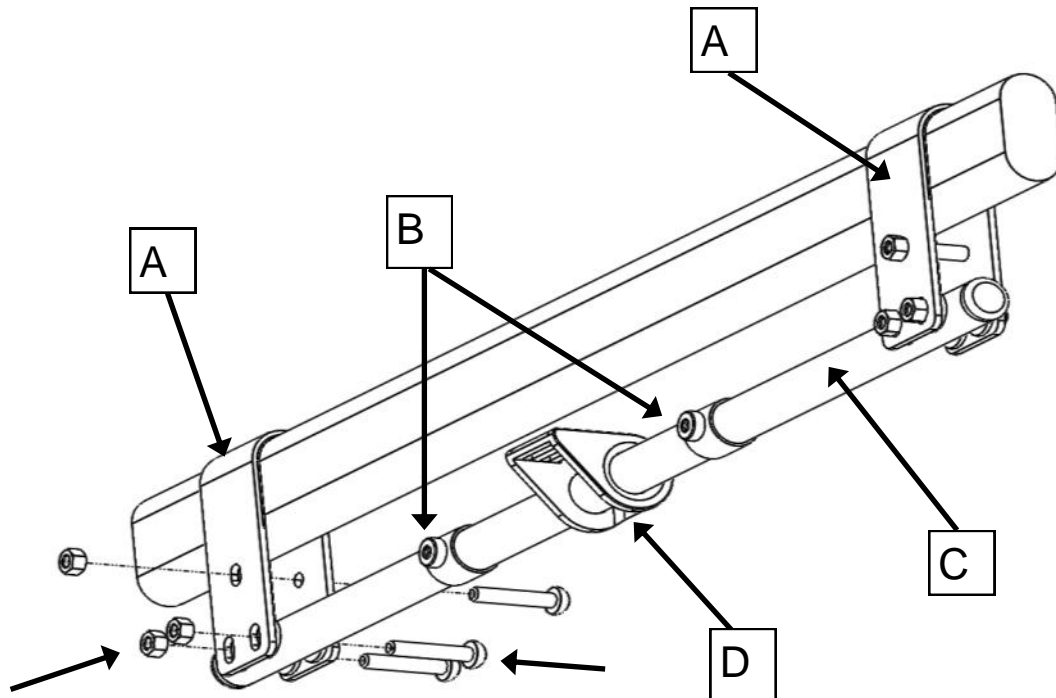

- Hook rail carriers, parts **A** over the main lower frame of the Grillo
- Ensure the stops **B** and slide **D** are on the rail as shown above.
- Align the rail **C** with the bolt holes insert the 2 lower bolts at each end and the upper bolts. Do not fully tighten until correct alignment is obtained.
- Thread the cuff straps through the tri-slots on the slide **D** as shown below right and adjust the length to suit the user.
- Adjust the rail to the correct gait alignment of the user and tighten fully.
- The stops **B** can be tightened after the correct stride has been obtained.
- The cuffs can be fasten around the ankle area using the hook and loop fastener.



It is recommended that time is taken to ensure correct set up to the users needs. The user should be supervised at all times when using the ankle cuffs on the Grillo. Should the equipment become damaged then it should not be used until repaired. Do not modify or use in any way other than for the intended use. Please check for suitable adjustment and security before use.

